

QUICKSTART GUIDE

RESCUE[®] Ready to Use

Rescue[™] Ready to Use Contact Times

DAILY USE	BACTERICIDAL FUNGICIDAL	VIRUCIDAL	NON-FOOD CONTACT SURFACE SANITIZER
1 Min	1 Min	1 Min	10 Sec
Surface should be visibly wet for contact time.			

Rescue[™] Ready to Use is excellent for daily use and outbreak situations. Rescue[™] Ready to Use has fast contact times. Make sure that the surface is visibly wet for the entire contact time.

For more information on a specific organism, use the Pathogen Claim Tool on the Rescue[™] Ready to Use page at www.LearnAboutRescue.com.

How to Use Rescue[™] Ready to Use




STEP 1 Prepare the bottle

Remove the lid and seal. Approved sprayers, foamers and empty bottles make dispensing onto a surface easy and ensures accurate and even distribution.




STEP 2 Remember to apply a workplace label to the solution container

Use a Rescue[™] Ready to Use workplace label and record the lot number and the date the product was decanted..

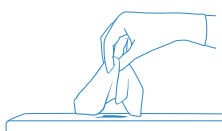


STEP 3 Pre-clean heavily soiled areas

Remove visible soil and pre-clean the area by working from the least contaminated area to the most contaminated area.

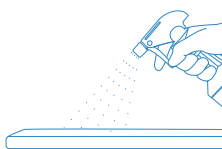

STEP 4 Squirt, foam or wipe Rescue[™] Ready to Use onto the surface

Spray the product 6-8 inches away from the surface making sure the surface area is visibly wet. Wettability and surface coverage are equally as important to ensure disinfection can be achieved in one application.

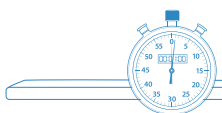

STEP 5 Allow surface to remain visibly wet for 1 minute

In order for disinfectants to be effective they must remain wet on the surface for their entire contact time. Be sure to follow the proper contact times for specific pathogens.



STEP 6 Wipe surface dry or allow to air dry

Use a damp, clean cloth to dry small areas, or allow to air dry.

Success & Troubleshooting Tips

Spray and Walk Away.

After spraying the product, walk away and come back after the appropriate contact time. Rescue[™] is non-toxic and is not harmful to inhale. The aerosolization of Rescue[™] may cause you to cough. You can also use a trigger foamer that will apply the product in a foam instead of a spray.

Wait Before Wiping Dry.

Always follow the required contact time when disinfecting a surface for a particular pathogen. Consider setting a timer and walking away until the timer goes off.

For more information visit:
Rescuedisinfectants.com

Virox Animal Health
2770 Coventry Road
Oakville, ON L6H 6R1 CANADA
Toll Free: 1-800-387-7578